



DiaSorin Italia S.p.A.
Via Crescentino snc - 13040 Saluggia (VC) - Italy
www.diasorin.com
Tel. +39.0161.4871



Changes: §4, §6;
Deletions: -

LIAISON® Measles IgM (REF 318820)

1. INTENDED USE

The LIAISON® Measles IgM assay uses chemiluminescence immunoassay (CLIA) technology for the qualitative determination of specific IgM antibodies to measles virus in human serum or plasma samples. It is intended to be used as an aid in the determination of serological status to measles virus.

The test has to be performed on the LIAISON® Analyzer family*.

2. SUMMARY AND EXPLANATION OF THE TEST

Measles is an acute viral illness caused by a morbillivirus of the paramyxovirus family and is one of the most easily transmitted diseases. Transmission is primarily by large droplet spread or direct contact with nasal or throat secretions from an infected person⁽¹⁾.

After infection, measles virus invades the respiratory epithelium of the nasopharynx and spreads to the regional lymph nodes. After two to three days of replication in these sites, primary viraemia widens the infection to the reticulo-endothelial system. Following further replication, secondary viraemia occurs five to seven days after infection and lasts four to seven days. During this viraemia, infection and further virus replication may occur in skin, conjunctivae, respiratory tract and other organs, including spleen, thymus, lung, liver, and kidney. Viraemia peaks 11-14 days after infection, and then declines rapidly over a few days⁽²⁾.

Prior to vaccine availability, measles was mostly a disease of childhood, but measles vaccination programs (part of measles, mumps, rubella [MMR] vaccination) have had a marked effect on the incidence of the disease and the complications associated with it. After prolonged periods of high vaccine coverage in developed countries, measles transmission now occurs mainly in people that have never been vaccinated and in older children who did not seroconvert following vaccination. Measles outbreaks can still occur in countries with high immunization coverage. Such outbreaks demonstrate an immunity gap in the population involved^(1, 3).

Clinically, the diagnosis of measles is supported if Koplik's spots are detected and if the rash progresses from the head to the trunk and out to the extremities⁽⁶⁾. The non-specific nature of the prodromal signs and the existence of mild cases, however, make clinical signs unreliable as the sole diagnostic criteria of measles disease. As disease prevalence falls, many medical practitioners are inexperienced in recognizing measles, increasing the need for laboratory serological method of distinguishing measles from other clinically similar diseases⁽⁴⁾.

Both IgM and IgG antibodies are synthesized during the primary immune response and can be detected in the serum within a few days of rash onset. IgM antibody levels peak after about seven to ten days and then decline rapidly, being rarely detectable after six to eight weeks. IgM is generally not detected in an immune individual following re-exposure to measles virus⁽⁵⁾. Re-exposure to the measles virus induces a strong anamnestic immune response with a rapid boosting of IgG antibodies, which prevents clinical disease⁽⁶⁾.

3. PRINCIPLE OF THE PROCEDURE

The method for qualitative determination of specific IgM to measles virus is an antibody capture chemiluminescence immunoassay (CLIA). IgG to human IgM (mouse, monoclonal) is used for coating magnetic particles (solid phase) and a recombinant measles virus antigen is linked to an isoluminol derivative (isoluminol-antigen conjugate). During the first incubation, IgM antibodies present in calibrators, samples or controls bind to the solid phase. During the second incubation, the measles virus antigen conjugate reacts with IgM already bound to the solid phase. After each incubation, the unbound material is removed with a wash cycle.

Subsequently, the starter reagents are added and a flash chemiluminescence reaction is thus induced. The light signal, and hence the amount of isoluminol-antigen conjugate, is measured by a photomultiplier as relative light units (RLU) and is indicative of measles virus IgM concentration present in calibrators, samples or controls.

*(LIAISON®, LIAISON® XL, LIAISON® XS)

4. MATERIALS PROVIDED

Reagent integral

Magnetic particles (1.6 mL)	[SORB]	Magnetic particles coated with IgG to human IgM (mouse, monoclonal), BSA, PBS buffer < 0.1% sodium azide.
Calibrator 1 (0.55 mL)	[CAL1]	Human serum/defibrinated plasma non-reactive for measles virus IgM, 0.2% ProClin™ 300, preservatives. The calibrator concentrations (Index) are referenced to an in-house antibody preparation.
Calibrator 2 (0.55 mL)	[CAL2]	<i>HuCAL® IgM Antibodies</i> (under license from MorphoSys AG. <i>HuCAL®</i> is a registered trademark of MorphoSys AG) reactive for measles virus, Casein, BSA, phosphate buffer, EDTA, detergents, preservatives, an inert blue dye. The calibrator concentrations (Index) are referenced to an in-house antibody preparation.
Specimen diluent (27 mL)	[DILSPE]	Casein, BSA, phosphate buffer, EDTA, detergents, preservatives, an inert blue dye.
Conjugate (13.5 mL)	[CONJ]	Recombinant measles virus nucleoprotein (obtained in baculovirus) conjugated to an isoluminol derivative, foetal calf serum, borate buffer, preservatives, an inert yellow dye.
Number of tests		50

All reagents are supplied ready to use. The order of reagents reflects the layout of containers in the reagent integral.

Materials required but not provided (system related)

LIAISON® XL Analyzer	LIAISON® Analyzer
LIAISON® XL Cuvettes ([REF] X0016).	LIAISON® Module ([REF] 319130).
LIAISON® XL Disposable Tips ([REF] X0015) or	–
LIAISON® Disposable Tips ([REF] X0055).	–
–	LIAISON® Starter Kit ([REF] 319102) or
LIAISON® XL Starter Kit ([REF] 319200) or	LIAISON® XL Starter Kit ([REF] 319200) or
LIAISON® EASY Starter Kit ([REF] 319300).	LIAISON® EASY Starter Kit ([REF] 319300).
–	LIAISON® Light Check 12 ([REF] 319150).
LIAISON® Wash/System Liquid ([REF] 319100).	LIAISON® Wash/System Liquid ([REF] 319100).
LIAISON® XL Waste Bags ([REF] X0025).	LIAISON® Waste Bags ([REF] 450003).
–	LIAISON® Cleaning Kit ([REF] 310990).

LIAISON® XS Analyzer
LIAISON® Cuvettes on Tray ([REF] X0053).
LIAISON® Disposable Tips ([REF] X0055).
LIAISON® EASY Starter Kit ([REF] 319300).
LIAISON® EASY Wash Buffer ([REF] 319301).
LIAISON® EASY System Liquid ([REF] 319302).
LIAISON® EASY Waste ([REF] X0054).
LIAISON® EASY Cleaning Tool ([REF] 310996).

Additionally required materials

LIAISON® Measles IgM controls (negative and positive) ([REF] 318821).

5. WARNINGS AND PRECAUTIONS

For *in vitro* diagnostic use.

All serum and plasma units used to produce the components provided in this kit have been tested for the presence of HBsAg, anti-HCV, anti-HIV-1, anti-HIV-2 and found to be non-reactive. As, however, no test method can offer absolute assurance that pathogens are absent, all specimens of human origin should be considered potentially infectious and handled with care.

6. SAFETY PRECAUTIONS

Do not eat, drink, smoke or apply cosmetics in the assay laboratory.

Do not pipette by mouth.

Avoid direct contact with potentially infected material by wearing laboratory clothing, protective goggles, and disposable gloves. Wash hands thoroughly at the end of each assay.

Avoid splashing or forming an aerosol. All drops of biological reagent must be removed with a sodium hypochlorite solution with 0.5% active chlorine, and the means used must be treated as infected waste.

All samples and reagents containing biological materials used for the assay must be considered as potentially able to transmit infectious agents. The waste must be handled with care and disposed of in compliance with the laboratory guidelines and the statutory provisions in force in each Country. Any materials for reuse must be appropriately sterilized in compliance with the local laws and guidelines. Check the effectiveness of the sterilization/decontamination cycle.

Do not use kits or components beyond the expiration date given on the label.

Pursuant to EC Regulation 1272/2008 (CLP) hazardous reagents are classified and labeled as follow:

REAGENTS:	CAL1	CONJ
CLASSIFICATION	Skin sens. 1A H317 Aquatic chronic 3 H412	Repr. 1B H360FD
SIGNAL WORD:	Warning	Danger
SYMBOLS / PICTOGRAMS:	 GHS07 – Exclamation mark	 GHS08 Health hazard
HAZARD STATEMENTS:	H317 May cause an allergic skin reaction. H412 Harmful to aquatic life with long lasting effects.	H360FD May damage fertility, May damage the unborn child
PRECAUTIONARY STATEMENTS:	P261 Avoid breathing dust/fume/gas/mist vapours/spray. P280 Wear protective gloves/protective clothing/eye protection/face protection. P273 Avoid release to the environment. P362 Take off contaminated clothing and wash before reuse.	P261 Avoid breathing dust/fume/gas/mist vapours/spray. P280 Wear protective gloves/protective clothing/eye protection/face protection. P202 Do not handle until all safety precautions have been read and understood. P308+313 If exposed or concerned: get medical advice/attention. P501 Dispose of contents/container in accordance with local/regional/national/international regulations
CONTAINS: (only substances prescribed pursuant to Article 18 of EC Regulation 1272/2008).	reaction mass of: 5-chloro-2-methyl-4-isothiazolin-3-one [EC no. 247-500-7] and 2-methyl-2H -isothiazol-3-one [EC no. 220-239-6] (3:1) (ProClin™ 300).	Acid Boric

Pursuant to EC Regulation 1272/2008 (CLP), **SORB** is labeled as EUH210 safety data sheets available on request. For additional information see Safety Data Sheets available on www.diasorin.com.

7. PREPARATION OF REAGENT INTEGRAL

Please note the following important reagent handling precautions:

Resuspension of magnetic particles

Magnetic particles must be completely resuspended before the integral is placed on the instrument. Follow the steps below to ensure complete suspension:

Before the seal is removed, rotate the small wheel at the magnetic particle compartment until the colour of the suspension has changed to brown. Gentle and careful side-to-side mixing may assist in the suspension of the magnetic particles (avoid foam formation). Visually check the bottom of the magnetic particle vial to confirm that all settled magnetic particles have resuspended. Carefully wipe the surface of each septum to remove residual liquid.

Repeat as necessary until the magnetic particles are completely resuspended.

Foaming of reagents

In order to ensure optimal performance of the integral, foaming of reagents should be avoided. Adhere to the recommendation below to prevent this occurrence:

Visually inspect the reagents, calibrators in particular (position two and three following the magnetic particle vial), to ensure there is no foaming present before using the integral. If foam is present after resuspension of the magnetic particles, place the integral on the instrument and allow the foam to dissipate. The integral is ready to use once the foam has dissipated and the integral has remained onboard and mixing.

Loading of integral into the reagent area

LIAISON® Analyzer

- Place the integral into the reagent area of the analyzer with the bar code label facing left and let it stand for 30 minutes before using. The analyzer automatically stirs and completely resuspends the magnetic particles.
- Follow the analyzer operator's manual to load the specimens and start the run.

LIAISON® XL and LIAISON® XS analyzers

- LIAISON® XL Analyzer and LIAISON® XS Analyzer are equipped with a built-in solid-state magnetic device which aids in the dispersal of microparticles prior to placement of a reagent integral into the reagent area of the analyzer. Refer to the analyzer operator's manual for details.
 - Insert the reagent integral into the dedicated slot.
 - Allow the reagent integral to remain in the solid-state magnetic device for at least 30 seconds (up to several minutes). Repeat as necessary.
- Place the integral into the reagent area of the analyzer with the label facing left and let it stand for 15 minutes before using. The analyzer automatically stirs and completely resuspends the magnetic particles.
- Follow the analyzer operator's manual to load the specimens and start the run.

8. REAGENT INTEGRAL STORAGE AND STABILITY

- **Sealed:** Stable at 2-8°C until the expiry date.
- **Opened on board or at 2-8°C:** Minimum stability eight weeks.
After this period, it is still possible to keep on using the reagent integral provided that the controls are found within the expected ranges.
- Use always the same analyzer for a reagent integral already opened.
- Use storage rack provided with the analyzer for upright storage of reagent integral.
- Do not freeze.
- Keep upright for storage to facilitate later proper resuspension of magnetic particles.
- Keep away from direct light.

9. SPECIMEN COLLECTION AND PREPARATION

Either human serum or plasma may be used. The anticoagulants potassium EDTA and heparin have been tested and may be used with this assay. Blood should be collected aseptically by venipuncture, allowed to clot, and the serum separated from the clot as soon as possible. Samples having particulate matter, turbidity, lipaemia, or erythrocyte debris may require clarification by filtration or centrifugation before testing. Grossly haemolyzed or lipaemic samples as well as samples containing particulate matter or exhibiting obvious microbial contamination should not be tested. Check for and remove air bubbles before assaying. If the assay is performed within seven days of sample collection, the samples may be kept at 2-8°C; otherwise they should be aliquoted and stored deep-frozen (-20°C or below). If samples are stored frozen, mix thawed samples well before testing. Ten samples with different reactivity were stored for seven days at 2-8°C and ten samples underwent six freeze-thaw cycles. The results showed no significant differences. The minimum volume required for a single determination is 170 µL specimen (20 µL specimen + 150 µL dead volume).

10. CALIBRATION

Test of assay specific calibrators allows the detected relative light unit (RLU) values to adjust the assigned master curve. Each calibration solution allows four calibrations to be performed.

Recalibration in triplicate is mandatory whenever at least one of the following conditions occurs:

- A new lot of reagent integral or of Starter Kit is used.
- The previous calibration was performed more than four weeks before.
- **LIAISON® and LIAISON® XL analyzers:** The analyzer has been serviced.
- **LIAISON® XS Analyzer:** after a technical intervention, only if required by the service procedure, as communicated by DiaSorin Technical support or representative.
- Control values lie outside the expected ranges.

LIAISON® Analyzer: Calibrator values are stored in the bar codes on the integral label.

LIAISON® XL Analyzer: Calibrator values are stored in the reagent integral Radio Frequency IDentification transponder (RFID Tag).

LIAISON® XS Analyzer: Calibrator values are stored in the reagent integral Radio Frequency IDentification transponder (RFID Tag).

11. ASSAY PROCEDURE

Strict adherence to the analyzer operator's manual ensures proper assay performance.

LIAISON® Analyzer. Each test parameter is identified via the bar codes on the reagent integral label. In the event that the barcode label cannot be read by the analyzer, the integral cannot be used. Do not discard the reagent integral; contact your local DiaSorin technical support for instruction.

LIAISON® XL and LIAISON® XS analyzers. Each test parameter is identified via information encoded in the reagent integral Radio Frequency IDentification transponder (RFID Tag). In the event that the RFID Tag cannot be read by the analyzer, the integral cannot be used. Do not discard the reagent integral; contact your local DiaSorin technical support for instruction.

The analyzer operations are as follows:

1. Dispense calibrators, controls or specimens into the reaction module.
2. Dispense specimen diluent.
3. Dispense coated magnetic particles.
4. Incubate.
5. Wash with Wash/System liquid.
6. Dispense conjugate into the reaction module.
7. Incubate.
8. Wash with Wash/System liquid.
9. Add the Starter Kit and measure the light emitted.

12. QUALITY CONTROL

LIAISON® controls should be run in singlicate to monitor the assay performance. Quality control must be performed by running LIAISON® Measles IgM controls

- (a) at least once per day of use,
- (b) whenever a new reagent integral is used,
- (c) whenever the kit is calibrated,
- (d) whenever a new lot of Starter Reagents is used,
- (e) to assess adequacy of performance of the open integral beyond eight weeks, or in agreement with guidelines or requirements of local regulations or accredited organizations.

Control values must lie within the expected ranges: whenever one or both controls lie outside the expected ranges, calibration should be repeated and controls retested. If control values obtained after successful calibration lie repeatedly outside the predefined ranges, the test should be repeated using an unopened control vial. If control values lie outside the expected ranges, patient results must not be reported.

The performance of other controls should be evaluated for compatibility with this assay before they are used. Appropriate value ranges should then be established for quality control materials used.

13. INTERPRETATION OF RESULTS

The analyzer automatically calculates measles virus IgM concentrations expressed as index value and grades the results. For details, refer to the analyzer operator's manual.

Calibrators and controls may give different RLU or dose results on LIAISON®, LIAISON® XL and LIAISON® XS, but patient results are equivalent.

On LIAISON® Analyzer

Assay range. 0.5 to 4 index value measles virus IgM.

The cut-off value discriminating between the presence and the absence of measles virus IgM is 1.0 index value. Sample results should be interpreted as follows:

Samples with measles virus IgM concentrations below 0.9 index value should be graded *negative*.

Samples with measles virus IgM concentrations ranging between 0.9 and 1.1 index value should be graded *equivocal*. *Equivocal samples must be retested in order to confirm the initial result. Samples which are positive at the second test should be considered positive. Samples which are negative at the second test should be considered negative. A second sample should be collected and tested no less than one to two weeks later when the result is repeatedly equivocal.*

Samples with measles virus IgM concentrations equal to or above 1.1 index value should be graded *positive*.

On LIAISON® XL and LIAISON® XS analyzers

The cut-off value discriminating between the presence and the absence of measles virus IgM is 1.0 index value. Sample results should be interpreted as follows:

Samples with measles virus IgM concentrations below 0.9 index value should be graded *negative*.

Samples with measles virus IgM concentrations ranging between 0.9 and 1.1 index value should be graded *equivocal*. *Equivocal samples must be retested in order to confirm the initial result. Samples which are positive at the second test should be considered positive. Samples which are negative at the second test should be considered negative. A second sample should be collected and tested no less than one to two weeks later when the result is repeatedly equivocal.*

Samples with measles virus IgM concentrations equal to or above 1.1 index value should be graded *positive*.

A positive result is generally indicative of acute infection. A negative result, however, does not always rule out acute measles virus, because the infection is in its very early stage and the patient may be still unable to synthesize measles virus specific IgM. If clinical exposure to measles virus is suspected despite a negative finding, a second sample should be collected and tested no less than one or two weeks later. An equivocal result is indicative either of recent infection or of past infection with long-lasting measles virus IgM. Serological data from detection of additional measles virus markers may provide useful information for clinical interpretation of results.

14. LIMITATIONS OF THE PROCEDURE

Assay performance characteristics have not been established when any LIAISON® measles test is used in conjunction with other manufacturers' assays for detection of specific measles serological markers. Under these conditions, users are responsible for establishing their own performance characteristics.

A skillful technique and strict adherence to the instructions are necessary to obtain reliable results.

Bacterial contamination or heat inactivation of the specimens may affect the test results.

Test results are reported qualitatively as positive or negative for the presence of measles virus IgM. However, diagnosis of infectious diseases should not be established on the basis of a single test result, but should be determined in conjunction with clinical findings and other diagnostic procedures as well as in association with medical judgement.

Specimens from patients receiving preparations of mouse monoclonal antibodies for therapy or diagnosis may contain human anti-mouse antibodies (HAMA). Such specimens may interfere in a monoclonal antibody-based immunoassay and their results should be evaluated with care.

Integrals may not be exchanged between analyzer types (LIAISON®, LIAISON® XL and LIAISON® XS). Once an integral has been introduced to a particular analyzer type, it must always be used on that analyzer until it has been exhausted. Due to traceability issues resulting from the above statement, patient follow-ups may not be concluded between analyzer types. These must be accomplished on one particular analyzer type (either LIAISON®, LIAISON® XL or LIAISON® XS).

15. SPECIFIC PERFORMANCE CHARACTERISTICS

15.1. Analytical specificity

Analytical specificity may be defined as the ability of the assay to accurately detect specific analyte in the presence of potentially interfering factors in the sample matrix (e.g., anticoagulants, haemolysis, effects of sample treatment), or cross-reactive antibodies.

Interference. Controlled studies of potentially interfering substances or conditions showed that the assay performance was not affected by anticoagulants (potassium EDTA, heparin, sodium citrate), haemolysis (up to 1000 mg/dL haemoglobin), lipaemia (up to 3000 mg/dL triglycerides), bilirubinaemia (up to 20 mg/dL bilirubin), or by up to six freeze-thaw cycles of samples.

Cross-reactions. The cross-reactivity study for the LIAISON® Measles IgM assay was designed to evaluate potential interference from antibodies to other organisms that may cause clinical symptoms similar to those of measles (hCMV, EBV, rubella virus, parvovirus B19, mumps virus, *Toxoplasma gondii*), from antibodies to other organisms that may cause infectious diseases (*Treponema pallidum*, HSV, VZV) as well as from other conditions that may result from atypical immune system activity (anti-nuclear autoantibodies, rheumatoid factor, HAMA or human anti-mouse antibodies). Samples for these studies were pre-screened with another commercially available measles virus IgM assay. Samples that were seronegative for measles virus IgG antibodies and seropositive for the cross-reactant were used in the study. The presence of potential cross-reactants in the samples was detected using CE-marked assays.

Clinical condition	Number of expected negative samples	LIAISON® positive or equivocal results
hCMV IgM antibodies	15	0
VZV IgM antibodies	6	0
Parvovirus B19 IgM antibodies	10	0
HSV IgM antibodies	5	0
Rubella virus IgM antibodies	8	0
EBV IgM antibodies	18	0
Mumps virus IgM antibodies	5	0
<i>Toxoplasma gondii</i> IgM antibodies	15	0
Anti-nuclear autoantibodies (ANA)	5	0
Rheumatoid factor (anti-Fc immunoglobulin)	5	0
<i>Treponema pallidum</i> antibodies	4	0
Human anti-mouse antibodies (HAMA)	7	0
Total	103	0

None of the specimens tested from the disease panel was positive. There was no conclusive evidence of cross-reactivity observed; however, due to the limited availability of certain samples, the possibility of cross-reactivity cannot be excluded. The results refer to the groups of samples investigated and are not guaranteed specifications, as differences may exist between laboratories and locations.

15.2. Precision with LIAISON® Analyzer

Different samples, containing different concentrations of specific analyte, were assayed to estimate repeatability and reproducibility of the assay (i.e., within- and between-assay variability). The results refer to the groups of samples investigated and are not guaranteed specifications, as differences may exist between laboratories and locations.

Repeatability. Twenty replicates were performed in the same run to evaluate in-house repeatability.

Repeatability	A	B	Positive control
Number of determinations	20	20	20
Mean (index value)	1.5	2.2	2.2
Standard deviation	0.12	0.14	0.10
Coefficient of variation (%)	7.7	6.2	4.3
Min. value (index value)	1.3	2.0	2.0
Max. value (index value)	1.7	2.6	2.3

Reproducibility. Twenty replicates were performed in different days (maximum of two runs per day) with three different lots of integral to evaluate reproducibility. The tests were performed in two sites, in house (site 1) and in an independent laboratory (site 2) using two different instruments.

Reproducibility - Site 1	A	B	Positive control
LOT No. 01			
Number of determinations	20	20	20
Mean (index value)	1.7	2.2	2.1
Standard deviation	0.14	0.16	0.12
Coefficient of variation (%)	8.4	7.3	5.9
Min. value (index value)	1.4	2.0	1.8
Max. value (index value)	1.9	2.5	2.3
LOT No. 02			
Number of determinations	20	20	20
Mean (index value)	1.3	1.9	1.8
Standard deviation	0.16	0.17	0.13
Coefficient of variation (%)	12.2	8.8	7.4
Min. value (index value)	1.1	1.6	1.6
Max. value (index value)	1.7	2.2	2.0
LOT No. 03			
Number of determinations	20	20	20
Mean (index value)	1.2	2.7	1.9
Standard deviation	0.16	0.25	0.29
Coefficient of variation (%)	13.3	9.4	13.5
Min. value (index value)	1.1	2.3	1.5
Max. value (index value)	1.6	3.1	2.2

Reproducibility - Site 2	A	B	Positive control
LOT No. 01			
Number of determinations	20	20	20
Mean (index value)	1.7	2.3	2.2
Standard deviation	0.23	0.25	0.17
Coefficient of variation (%)	13.2	11.1	7.6
Min. value (index value)	1.2	1.7	1.9
Max. value (index value)	2.0	2.6	2.6
LOT No. 02			
Number of determinations	20	20	20
Mean (index value)	1.7	2.2	2.2
Standard deviation	0.15	0.28	0.24
Coefficient of variation (%)	8.7	12.4	11.2
Min. value (index value)	1.3	1.5	1.6
Max. value (index value)	1.9	2.6	2.7
LOT No. 03			
Number of determinations	20	20	20
Mean (index value)	1.4	2.3	2.1
Standard deviation	0.21	0.24	0.29
Coefficient of variation (%)	13.3	10.1	13.5
Min. value (index value)	1.2	1.9	1.6
Max. value (index value)	1.9	2.7	2.6

15.3. Precision with LIAISON® XL Analyzer

Different samples, containing different concentrations of specific analyte, were assayed to estimate repeatability and reproducibility of the assay (i.e., within- and between-assay variability). The variability shown in the tables below did not result in sample misclassification.

Repeatability. Twenty replicates were performed in the same run to evaluate repeatability.

Repeatability	1	2	Positive control
Number of determinations	20	20	20
Mean (index value)	1.7	2.6	2.3
Standard deviation	0.03	0.04	0.07
Coefficient of variation (%)	1.8	1.6	2.9
Min. value (index value)	1.7	2.6	2.1
Max. value (index value)	1.8	2.7	2.4

Reproducibility. Twenty replicates were performed in different days (maximum of two runs per day) with three different lots of integral to evaluate reproducibility.

Reproducibility	1	2	Positive control
LOT No. 01			
Number of determinations	20	20	20
Mean (index value)	1.7	2.4	1.9
Standard deviation	0.11	0.11	0.13
Coefficient of variation (%)	6.8	4.6	6.5
Min. value (index value)	1.5	2.3	1.8
Max. value (index value)	2.0	2.7	2.4
LOT No. 02			
Number of determinations	20	20	20
Mean (index value)	1.6	2.4	2.0
Standard deviation	0.14	0.16	0.10
Coefficient of variation (%)	8.8	6.7	5.0
Min. value (index value)	1.4	2.2	1.9
Max. value (index value)	1.9	2.7	2.2
LOT No. 03			
Number of determinations	20	20	20
Mean (index value)	1.4	2.4	3.1
Standard deviation	0.07	0.07	0.19
Coefficient of variation (%)	4.8	3.1	6.2
Min. value (index value)	1.3	2.3	2.9
Max. value (index value)	1.5	2.6	3.6

15.4. Precision with LIAISON® XS Analyzer

A five day precision study was conducted on three LIAISON® XS Analyzer to verify the precision with the LIAISON® Measles IgM Assay. The CLSI document EP15-A3 was consulted in the preparation of the testing protocol.

A coded panel comprised of frozen samples was used for the study.

The samples could be prepared by pooling samples with similar title in order to represent negative, borderline and positive levels.

The LIAISON® Control Measles IgM set was also included in the five day study.

The coded panel was tested on three LIAISON® XS Analyzers, in six replicates in a single run per day, for 5 operative days.

The mean Index value, standard deviation, and coefficient of variation (%CV) of the results were computed for each of the tested specimens for each of the instruments and across instruments.

Repeatability. Ninety replicates were performed in the same test to evaluate repeatability. 7 serum samples containing different concentration of analyte and kit controls were assayed in 6 replicates per day, over 5 operating days, on 3 units and one reagent lot.

Repeatability	3	4	5	6	7	8	9
Number of determinations	90	90	90	90	90	90	90
Mean (index value)	0.41	1.0	1.3	3.1	3.3	4.5	4.2
Standard deviation	0.009	0.019	0.036	0.082	0.064	0.11	0.11
Coefficient of variation (%)	2.3	1.8	2.7	2.6	1.9	2.5	2.5
Min. value (index value)	0.36	0.87	1.1	2.5	2.6	3.9	3.5
Max. value (index value)	0.46	1.2	1.6	3.8	4.0	5.4	5.4

Reproducibility. Ninety replicates were performed in different days (one run per day) to evaluate reproducibility. 7 serum samples containing different concentration of analyte and kit controls were assayed in 6 replicates per day, over 5 operating days, on 3 units and one reagent lot.

Reproducibility	3	4	5	6	7	8	9
Number of determinations	90	90	90	90	90	90	90
Mean (index value)	0.41	1.0	1.3	3.1	3.3	4.5	4.2
Standard deviation	0.018	0.031	0.072	0.186	0.209	0.190	0.237
Coefficient of variation (%)	4.4	3.0	5.4	5.9	6.3	4.2	5.6
Min. value (index value)	0.36	0.87	1.1	2.5	2.6	3.9	3.5
Max. value (index value)	0.46	1.2	1.6	3.8	4.0	5.4	5.4

15.5. High-dose saturation effect

Whenever samples containing extremely high antibody concentrations are tested, the saturation effect can mimic concentrations lower than real. However, a well-optimized two-step method excludes grossly underestimated results, because the analytical signals remain consistently high (saturation curve).

Analysis of saturation effect was evaluated by testing four high-titred samples positive for measles virus IgM. All samples resulted in high concentration values as expected, indicating no sample misclassification.

15.6. Diagnostic specificity and sensitivity

Diagnostic specificity and sensitivity were assessed by testing 532 specimens collected from a European laboratory routine. The specimens were tested by a comparison method and consensus with additional clinical and serological data was applied to define the expected results. Four specimens were unresolved by the reference method and therefore were not included in the data analysis.

No positive, no equivocal and 376 negative results were observed in the expected negative population studied - diagnostic specificity: 100% (376/376) (95% confidence interval: 99.0-100.0%).

Three negative, two equivocal and 147 positive results were observed in the expected positive population studied - diagnostic sensitivity: 96.7% (147/152) (95% confidence interval: 92.5-98.9%).

REFERENCES

1. T.H. TULCHINSKY et al.
Measles control in developing and developed countries: the case for a two-dose policy.
Bull. World Health Organ., **71** (1) : 93-103 (1993).
2. R.D. FEIGIN, J.D. CHERRY
Textbook of Pediatric Infectious Diseases, R.D. Feigin, J.D. Cherry eds., W.B. Saunders, 4th edition, p. 2054-2074 (1997).
3. Measles, in Epidemiology and Prevention of Vaccine-Preventable Diseases "The Pink Book".
Centers for Disease Control and Prevention, Atlanta, p. 115-133 (2004).
4. WHO-recommended standards for surveillance of selected vaccine-preventable diseases.
World Health Organization, WHO/V&B/03.01 (2003).
5. F. CUTTS, I. MODULE
Measles, in The Immunological Basis for Immunization, World Health Organization, Geneva (1993).
6. S.C. REDD, L.E. MARKOWITZ et al.
Measles vaccine, in Vaccines, S.A. Plotkin, W.A. Orenstein eds., W.B. Saunders, Philadelphia, p. 222-266 (1999).

200/007-028, 12 - 2023-11