



Changes: Update of Legal Manufacturer name;
Deletions: -

LIAISON® XL Disposable Tips ([REF] X0015)

1. INTENDED USE

LIAISON® XL Disposable Tips ([REF] X0015) are In Vitro Diagnostic Accessories and are intended to be used with the LIAISON® XL Analyzer to allow the LIAISON® XL Assays execution.

Tips take and dispense the proper amount of sample.

Both disposable tips and trays cannot be re-used and they shall be discarded upon completion of the assay procedure.

Use only DiaSorin approved disposable tips.

2. MATERIALS PROVIDED

- 1 shipping box contains 12 user boxes.
- Each user box contains 6 disposable tip trays with 96 disposable tips per tray.
- 576 tips per user box, totalling 6912 single disposable tips per shipping box.
- For additional details such as proper test performance of LIAISON® XL assays consult instructions for use of the assay to be performed.

3. HANDLING

For the description of the proper handling procedure, refer to the LIAISON® XL User Manual.

4. STORAGE

- Unopened at 0-30°C in the original box.
- Do not expose to sunlight or humidity.
- After use, reseal the box to protect the tips from contamination.
- Do not use after the expiration date given on the label.

5. WARNINGS AND PRECAUTIONS FOR USERS

- Observe the normal precautions required for handling all laboratory reagents.
- All waste material should be handled with care and disposed of in accordance with the laboratory guidelines and statutory provisions in force in each Country.
- If material packaging has been damaged upon delivery, please notify DiaSorin representative immediately. In this case, the usage of tips shall be avoided before any approval from DiaSorin.
- Any disposable tips whose cleanness and integrity cannot be ensured (e.g. fallen to the ground) are not to be utilized on the system and should be discarded immediately.
- Only tips from trays coming from user boxes have to be used on the LIAISON® XL Analyzer. It is not allowed to load the tips manually in order to prevent contamination and allow proper lot traceability.
- Tips must not be reused.
- Tips trays must not be reused.
- For Professional Use Only.

For EU only: please be aware that any serious incident that has occurred in relation to this IVD medical device should be reported to DiaSorin and the competent authority of the EU Member State in which the user and/or patient is established.